IMPORTANTE

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IMPORTANT

Product Description

CONTACTA (methafilcon) Daily Disposable Lenses are visibility tint, with UV blocker, supplied in a sterile state, immersed in buffered saline solution in sealed containers. These soft (Hydrophilic) sterile contact lenses are intended for daily wear. Daily disposable lenses are for single use only. Ensure healthy vision and comfortable wear by adhering strictly to the advice of your eye care practitioner.

Lens Specification

Base Curve 8,60 mm
Diameter 14,20mm
Center Thickness 0,04mm to 0,30mm
Powers -0,50D to -6,00D (0,25D/step)
-6,50D to -8,00D (0,50D/step)
Water Content 57 ± 2 %
Refractive Index 1,407 ± 0,005
Transmittance ≥ 90 %

Indications (Uses)

CONTACTA is intended to be used for daily wear for correction of refractive ametropia (myopia, hyperopia) in aphakic or non-aphakic people with non-diseased eyes who may have 2,00D of astigmatism or less.

Contraindications (Reasons not to use)

DO NOT USE the lenses when any of the following conditions exist: * Acute and subacute inflammation or infection of the anterior chamber of the eye, * Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids, * Severe insufficiency of lacrimal secretion (dry eyes), * Corneal hypoesthesia (reduced corneal sensitivity), if non-aphakic, * Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses, * Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses, * Any active corneal infection (bacterial, fungal, or viral), * If eye becomes red or irritated.

Precautions

1, Visit eye doctor or practitioner and approach training before initial wearing to ensure patient's abilities of wearing and removing properly of the contact lenses, 2, Always make regular visits to your eye care practitioner and follow the recommendations, 3, Daily disposable lens is for single use only. Do not reuse because it may cause eyes injury, 4, The lens is intended for daily wear not for extended wear, 5, Inform your health care doctor as being a contact lens wearer.

Side Effects

The patient should be informed that the following problems may occur: * Eyes stinging, burning, itching (irritation), or other eye pain, * Comfort is less than when lens was first placed on eye, * Feeling that something is in eye such as foreign body or scratched area, * Excessive watering (tearing) of the eyes, * Unusual eye secretions, * Redness of the eyes, * Reduced sharpness of vision (poor visual acuity),

* Blurred vision, rainbows, or halos around objects, * Sensitivity to light (photophobia), * Dry eyes.

When any of the above eye reaction occurs, **do remove the lenses and seek immediate professional identification and prompt treatment to avoid serious eye damage.**

Serious Incident Reporting
If the patient experiences any serious incident associated with the use of CONTACTA Daily Disposable Lenses (methafilcon), contact a licensed eye care professional and please notify Fidia farmaceutici S.p.A. in Italy: tel. +39 049 8232111, www.contactalens.com

They will report such important information to the manufacturer or local competent authority.

Instruction for Use
Always wash and dry your hands before handling your lenses. Initially remove lens from packaging and check that the seal of the packaging is intact and the lens is immersed in fluid. Do not use if the packaging or the lens is not intact.

Insertion: 1, Ensure that the lens is not inside out. Place the lens on tip of right index finger and verify the shape (fig. 1 and 2), 2, Blink a few times in case you do not see clear immediately, 3. Hold up the upper eyelid using left hand's index and middle fingers while holding down the lower eyelid with your right hand's middle finger. You can now place the lens onto the eye (fig. 3), 4. Repeat the procedure for the left eye.

Removal: 1. Wash and dry your hands, 2. Look up and while holding your right lower lid down, gently pinch the bottom edge of the lens and lift out of your eye with the pads of your index finger and thumb, 3. Repeat the procedure for the left eye (fig. 4), 4. If the lens sticks (stops moving) on your eyes, you should apply 1 to 2 drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. (If no movement of the lens continues after 5 minutes, you shall immediately consult the eye care practitioner.

Warning and Caution
* Do remember that the health of your eyes is essential for good sight. Don't put it at risk through negligence or vanity, * Do remove your lenses before going to sleep, * Do attend your routine follow up appointments. These are essential to monitor the health of your eyes. Symptom-free wear does not necessarily mean that your lenses are not causing undesirable side effects, * Do comply with your eye care practitioner's guide to wearing schedules. Over wear can have serious consequences, * Do keep your nails short. Long nails harbour bacteria and a scratched eye is vulnerable to infection, * Do replace lenses if they become damaged or scratched, or if they have deposits which cannot be removed, * Do wear make up if you wish, but insert your lenses first. Remove your lenses before you take off your make up. Fiber free mascara and oil free types are best, * Do contact your eye care professional before using any medicine, * Don't expose your contact lenses to any water (e.g. tap, bottled, distilled water, shower, hot tubs, participating in water activities), * Don't share your contact lenses with others, * Don't ever use household products (e.g. disinfectants) with your lenses, They can seriously damage your eyes, * Don't change lens type, parameters without consulting your eye care practitioner, * Don't use saliva to wet your lenses. It is tempting with potentially harmful bacteria. It is not true, that your own bacteria are safe, * Don't allow hair lacquer or other cosmetic aerosols to come into contact with your lenses, * Don't wear lenses in the presence of noxious or irritating fumes.

Storage and handling conditions
Recommended storing and operating at room temperature, Transporting with careful attention to high temperature (max. 45°C).

The following symbols may appear on the labelling or on the package:

SYMBOLS				
BC / B.C.	Base Curve	LS / DIA	Diameter	Manufacturer
PWR/SPH/DIOP	Dioptre / Spherical Power			
EC REP	Authorized representative in the European Community / European Union	LOT	Batch code	
EXP	Do not use if package is damaged and consult instructions for use	STERILE	Sterilized using steam or dry heat	
EXP	Use-by date / Expiry date	EXP	Consult instructions for use	
CE 1639	CE marking for conformity	MD	Medical device	
EXP	Do not re-use	Date of manufacture	Caution	Importer
EXP	Single sterile barrier system	EXP	UV Blocking	
EXP	Recycling	EXP	Do not waste into the environment	

YOUR EYES ARE PRECIOUS

Please read about your new contact lenses' Instruction for Use.

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